

Airon

Pneumatic Ventilation + CPAP


Safe-T-Vent[®] Nebulizer

Airon Part Number 58250

Packaged 10 per box

Aerosol Medication Nebulizer and Tee Set for use in CPAP systems and Ventilators

- High performance nebulizer with low particle size for effective drug delivery
- Holds CPAP and ventilation pressures when nebulizer is removed
- Operates in wide range of vertical orientations
- Works with all Airon patient breathing circuits and most other ventilator circuits
- Easily fits CPAP mask as well as ET-Tube

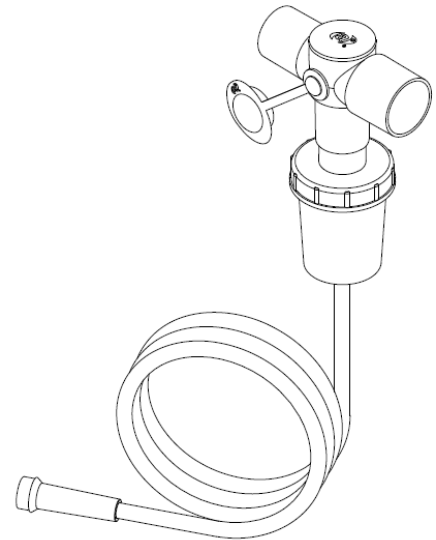

The **Safe-T-Vent** is designed to nebulize medications for inhalation in conjunction with the **pNeuton Ventilator / MACS CPAP System** and other positive pressure breathing devices. The **Safe-T-Vent** nebulizer can also be used as a stand alone medication nebulizer.

This unique nebulizer system includes a spring loaded seal built into the **Safe-T-Vent**; the seal allows the nebulizer to be removed while maintaining positive pressure. When placed in a patient breathing circuit for a ventilator or CPAP system, the patient will always receive the desired CPAP or positive pressure breaths. Pressures are maintained even when the nebulizer is removed for filling or accidentally falls out. The **Safe-T-Vent** Nebulizer removes the concern of losing pressure while providing worry free patient pressure delivery when giving aerosol medications.

The **Safe-T-Vent** Nebulizer is a disposable, single use device.

Specifications:

- Mass Median Aerodynamic Diameter (MMAD) – 2.4 microns
- Tee connections – 22 mm OD X 15/22 mm patient connection
- Optimum flow rate – 5 - 9 L/min



Airon



LATEX
FREE



CE
0413

Airon Corporation
751 North Drive, Unit 6, Melbourne, Florida 32934 USA
telephone +1-321-821-9433 fax +1-321-821-9443
CD-O-077 Rev A